



Creating a New Patient




Press the <<New Patient>> icon:



Enter the patient information in the <<New Patient>> window then select **Create Patient**. Red fields are mandatory.




The <<Order>> window will appear. Select the desired <<Order>> and press **Save**.

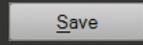


Click  and the new patient will appear in the patient database.

ID	Lastname	Firstname	Date of birth
DOEJO0000	Doe	John	01/01/1950
DOEJE0000	Doe	Jenna	01/01/1950



Creating a New Order



Press the <<New Order>> icon:



Select the desired work order from the drop down list in the order window and press **Save** to return to the patient database.



Click  and the order will appear under the patient worklist.



Press **Start Scan** to begin acquiring images.



The <<Eye Data>> window will appear. Enter the c-curve values or leave them at default and press <<OK>>.